



## Siderail Adjustment

**To lower rail:** push button and rotate away from center of bed.



**Installation and Removal:** pull pin and pull siderail away from frame.



## Expanding/Retracting Bed Deck

Lift handle to unlock while pulling out or pushing in section.


## Hand Control

Engage brake casters before adjusting bed.

1. Hold to flatten bed for CPR
2. Cardiac Chair position
3. Head/foot section
4. Bed height
5. Trend/reverse Trend
6. Control Buttons
7. Lockout Switches
8. Illuminated Buttons
9. Master Lockout Switch



## Footboard Controls

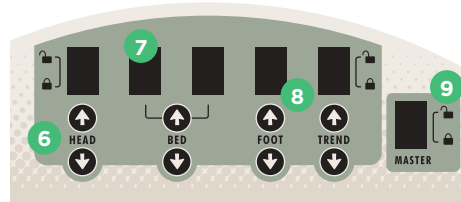

Located at foot end of bed. Controls include individual lockout functions and master lockout switch. Locking out specific or all functions will disable affected functions on both hand and foot control.

## Code Situation

1. Unlock casters and pull bed from wall.
2. Engage brakes, remove headboard and lower siderails for patient access.
3. If using air surface, press CPR button or turn CPR dial on blower and open CPR valve at head of mattress (if installed).
4. All beds: press and hold CPR button on hand control to flatten bed. New beds: pull and hold CPR handle located between siderails (both sides of bed).
5. Proceed with facility protocols for resuscitation.

## Components



1. Hand Control
2. Head Angle Indication
3. Brake Casters
4. Lock and Steer Casters
5. Foot Controls
6. RSS (some models)

## Troubleshooting

### If bed fails to function:

1. Verify all functions on footboard control panel are unlocked.
2. Verify master switch is unlocked.
3. Verify power cord is plugged into working AC outlet and not cut, frayed or damaged.
4. Inspect patient hand control for damage and ensure it is properly connected.

### If individual function does not operate:

1. Verify all functions on footboard control panel are unlocked.
2. Verify master switch is unlocked.

### If bed will not operate on battery power:

Verify patient hand control is properly connected.



## Battery Backup

This bed is equipped with an emergency battery backup that will charge when plugged into an AC outlet.

## Casters

- To **ENGAGE** brake, press lever down.
- To **DISENGAGE**, raise lever.



# REST SECURE SYSTEM™ (some models)

## System Setup

1. Plug into wall AC power.
2. Plug in Nurse Call interface.
3. Plug in Hand Control.

## Zero Scale

Press and hold **ZERO** for 3 seconds.

## Add/Remove Items

Press **UP** or **DOWN**.

Select **Add/Remove Items** and press **ENTER**.

Press **ENTER** after changes are made.

## Weighing Patient

Press **ENTER**.

View patient weight on monitor.

System will enter standby after 60 seconds of inactivity.

## To Change Units

Press **UP** or **DOWN** button and select scale settings.

Select **CHANGE LBS/KGS** and press **ENTER**.

Select **CHANGE TO LBS** or **CHANGE TO KGS** and press **ENTER**.

## Bed Exit Alarm

### Activate Alarm

Patient must be in bed prior to activation.

Press **BED EXIT** to select desired sensitivity.



#### High Sensitivity:

Detects patient movement. Do not use with alternation surface.



#### Medium Sensitivity:

Detects patient attempting exit. Do not use with rotation surface.



#### Low Sensitivity:

Detects when patient has exited the bed.

### Disable Alarm

Press **ENTER** and hold for 3 seconds to silence and disable alarm.

Bed exit alarm must be reactivated after being disabled.

### Reset Alarm

Place patient back in bed. Press **ENTER** to reset the alarm and restart the monitoring process.

## Adjusting Alarm Volume


Press **VOLUME** button to select desired sound level.

**NOTE:** When bed exit alarm/protocol timer are disabled, the sound level indicator lights will be off.

## Protocol Timer

Press **TIMER** button to select desired time: 1, 2 or 4 hours.

**To disable:** Press timer button until indicator lights are off.

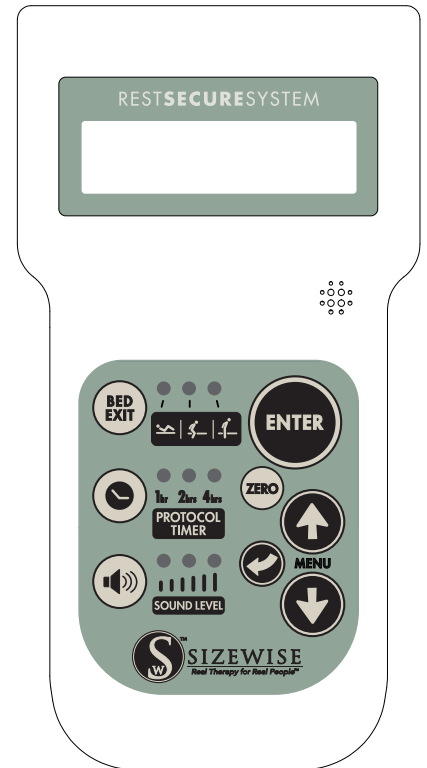

## ADDITIONAL RESOURCES

Scan for more Evolution® Line Training/Support Materials



- Printable Quick Reference Guide
- User Manual(s)
- Instructional Videos

This guide is not meant to replace the product user manual. For additional product support, call **800-814-9389**.

OPS-0008-0323 Replaces legacy QRGs 27650002, 27650000, 27650001 and 27650027. See user manual(s) for full content.

