



# Legacy/APM™ Control Unit

## QUICK REFERENCE GUIDE

---

### MATTRESS INFLATION/AUTOFIRM



Press **AUTOFIRM** button to fully inflate surface.  
Transfer patient to surface.

### ALTERNATION THERAPY

#### Mode

1. Press **MODE** button until Alternation LED shows.
2. Set cycle time by pressing the **CLOCK** button repeatedly until desired time shows (10, 15, 20, or 25).
3. Adjust patient comfort by using the **SOFT** and **FIRM** buttons. Each LED represents approximately 5.75 mmHg, ranging from 10 mmHg to 50 mmHg.

### STATIC (IMMERSION) THERAPY

#### Mode

1. Press **MODE** until Static LED shows.
2. Adjust patient comfort by using the **SOFT** and **FIRM** buttons. Each LED represents approximately 5.75 mmHg, ranging from 10 mmHg to 50 mmHg.

### ALARM SILENCE



Alarm indicates a low pressure reading.  
Check power supply and press **ALARM** button to silence.

This guide is not meant to replace the product user manual.  
For additional product support, call **800-814-9389**.