



OPERATING INSTRUCTIONS

1. Locate both power switches on left side of control unit.
2. Turn both switches to **ON** position.



Mattress Inflation/Autofirm

1. Press **AUTOFIRM** button to fully inflate surface.
2. Transfer patient to surface. **AUTOFIRM** times out after 10 minutes and resets to previous setting. To manually exit, press **AUTOFIRM** a second time.

**PATIENT
RIGHT**



Rotation Therapy

1. Choose desired turn by pressing corresponding button:
 - **Right or Left Turn:** maintains a turning cycle only to the selected side.
 - **Full Turn:** provides Continuous Lateral Rotational Therapy.

**PATIENT
LEFT**



2. After selecting desired turn, press **CYCLE TIME +/-** buttons until desired time is reached. If using left or right turn as a holding cycle, press **RIGHT** or **LEFT** button until Hd (hold) shows on screen. Patient will be held on desired side.

**FULL
TURN**



**TRAINING
MODE**



3. After selecting therapy, pressing **TRAINING MODE** will slowly acclimate patient to therapy over a period of 60 minutes.



Immersion Therapy (Static)

1. Press **STATIC** button.
2. Set firmness using **FIRM/SOFT** buttons based on patient comfort and clinical discretion.



Pulsation Therapy

1. Set firmness using **FIRM/SOFT** buttons based on patient comfort and clinical discretion.
2. Press **PULSATE** button.
3. Press **PULSATE** button until desired time is reached (3, 5 or 10 min).

PATIENT TRANSPORT

1. Activate **Autofirm** mode for 3-5 minutes to ensure mattress is completely inflated.
2. Turn both power switches off (left side of control unit) and unplug from wall.
3. Mattress will slowly deflate. As patient sinks into mattress, patient will rest on foam transport pad.

OTHER FEATURES



Fowler

Increases airflow when head section of bed is elevated. Newer models include an Auto Fowler feature.



Lockout

Locks control panel (auto after 3 min). To unlock, press and hold for 3 sec.



Alarm Reset

Light remains illuminated until problem is corrected.



Firm/Soft

Increases **(FIRM)** or decreases airflow **(SOFT)** to create a firmer or softer surface.

CPR/QUICK DEFLATION

1. **CPR control is on the front of control unit.** Depending on model, push red button or turn dial clockwise to activate CPR. Alarm will sound.
2. Locate red CPR valve at head of mattress. Turn valve for quick air release. **-OR-** Locate hose connection to control unit (all models). Disconnect for quick air release.



This guide is not meant to replace the product user manual.
For additional product support, call **800-814-9389**.