

Tom 2

Quick Reference Guide



Digital Training Resources

Scan with your phone's camera or visit [sizeWise.com/qr/tom2](https://www.sizeWise.com/qr/tom2)



Supervisor Control Panel

Go Button

1. Press **GO BUTTON** to activate controls.
2. After three minutes of inactivity, bed will need to be reactivated.

Stop Button

1. Press **STOP BUTTON** to immediately interrupt all bed movements.

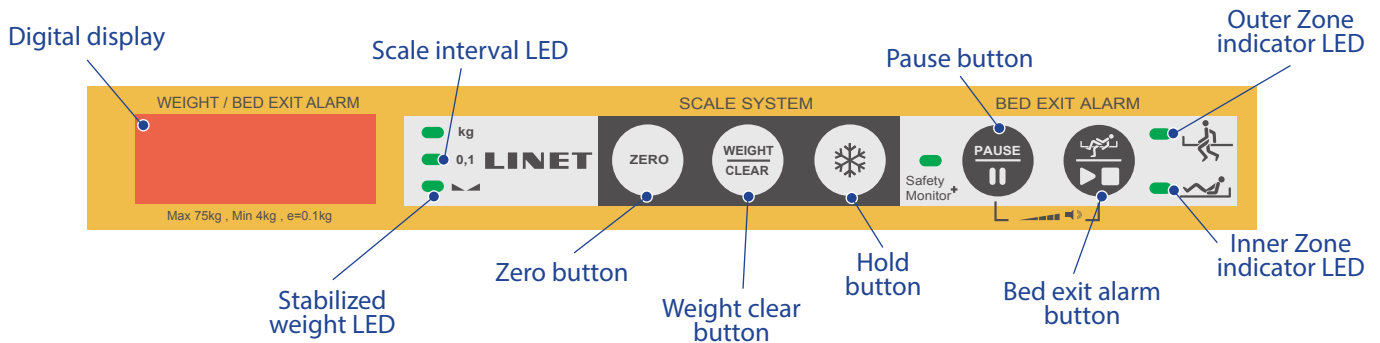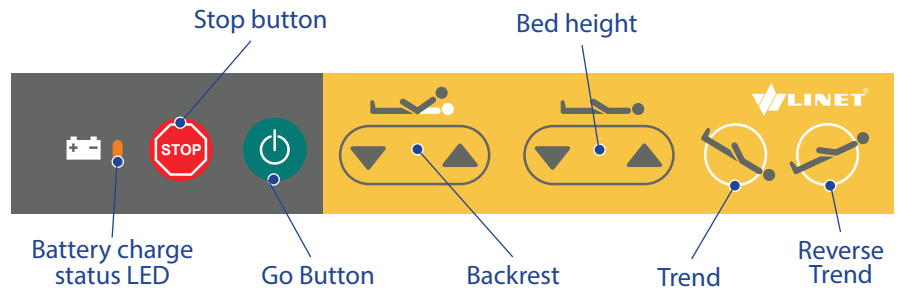

Operating Bed Scales

Note: Press any button to wake scale system.

Zeroing Scale

1. Place all items on bed needed for patient (sheets, pillows, blankets, etc.).
2. Press and hold **ZERO** button until display window flashes.
3. Press **ZERO** button again to confirm. Do not lean on footboard during zero process.

Weighing Patient

1. With patient in bed, press **WEIGHT/CLEAR** button to display patient weight.

Adding/Removing Items

1. Press **HOLD** button until digital weight display reads "hold" and LED glows green.
2. Add or remove items.
3. Press **HOLD** again to resume normal weight mode. Hold mode is only possible when weight is stable.

Bed Exit Alarm

Arming Bed Exit

1. With patient in bed, press and hold **BED EXIT ALARM** button until you hear a beep. The Bed Exit Alarm will be armed in the zone indicated by the LED.
3. To change zones, press **BED EXIT ALARM** button again until LED changes to illuminate desired zone.

Disarming Bed Exit

1. Press and hold **BED EXIT ALARM** button until both indicators are off.

Pausing Bed Exit

1. Press **PAUSE** for two seconds.
2. "P 15" is shown on display, indicating a pause period for 15 minutes. A decreasing number next to "P" indicates countdown. While Bed Exit is paused, no acoustic alarm is activated.

Setting Alarm Volume

1. Press **PAUSE** and **BED EXIT ALARM** simultaneously until desired volume is reached.


Siderails

To Lower

1. Pull up on siderail while grasping the handles with an underhand grip.
2. Press and hold both handles inward.
3. Gently lower siderail to desired position, then release. Ensure siderail clicks into position.

To Raise

1. Grasp siderail bars and lift up into desired position.
2. Ensure siderail clicks into position.

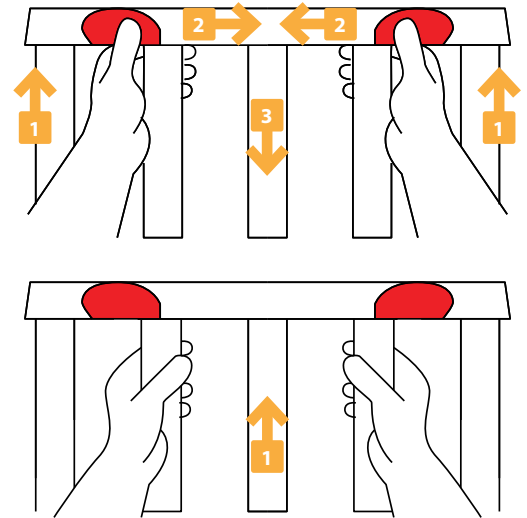

Patient Access Door


Never leave patient unattended while access door is open.

To Open

1. Pull and hold locking pin toward you.
2. While still holding locking pin, lift patient access door up to highest position (a magnet will hold it in place).

To Close

1. Push patient access door down carefully until it latches into locked position.
2. Ensure locking pin is secured properly.

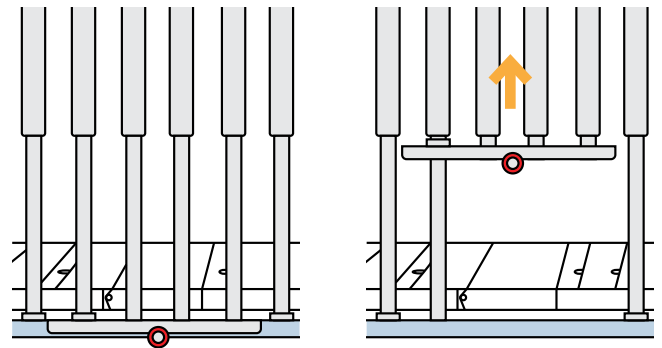



Bed End Panels

To Remove


1. Release both magnetic locks at foot end of bed by sliding inward.
2. Grasp and remove panel with both hands.
3. Lift panel up, rotate slightly toward bed, and remove.

To Install


1. Align panel with guides at top and bottom of bed end.
2. Slide panel down. Ensure panel "clicks" into position and locks at top and bottom of panel are secure.

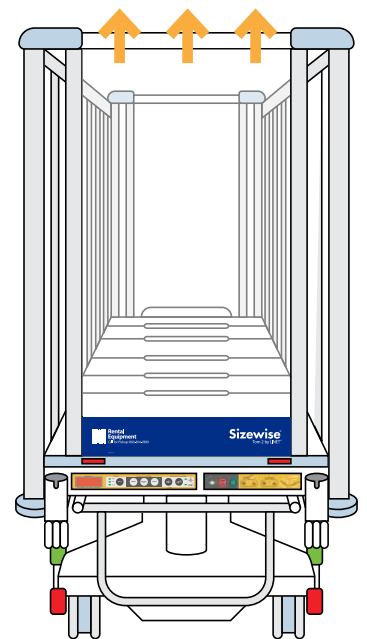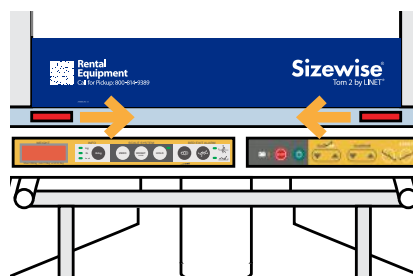


Casters

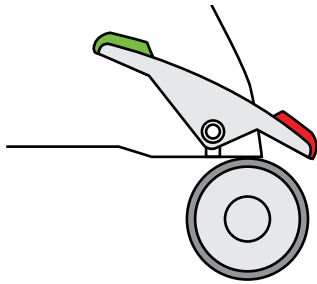

Braked
All casters are braked.

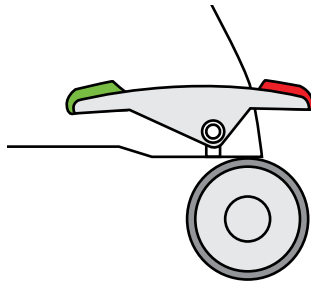

Unrestricted movement
Casters are unbraked, bed can move in any direction.

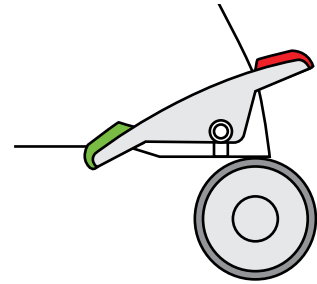


Forward Movement
Bed moves straight ahead.

Pediatric Pulse™

Note: Pediatric Pulse is an optional low air loss support surface available with Tom 2.


Blower Functions

See Quick Reference Label on front of the blower unit for instructions.

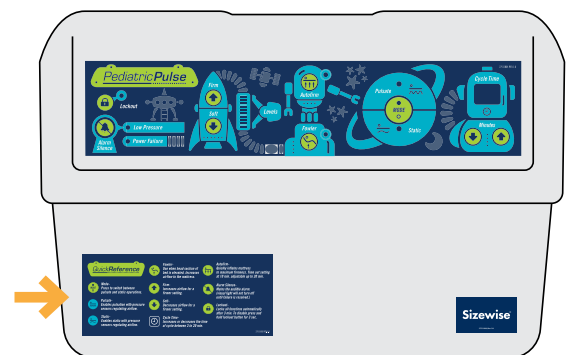

CPR/Code Situation

1. Unlock casters and pull bed from wall.
2. Engage brakes and lower siderails for patient access.
3. For air Support Surface Quick Deflation or CPR:
 - a. Turn power off on control unit.
 - b. Twist quick release connector on mattress to open
 - c. Remove hoses from control unit
4. Activate CPR function pull **CPR LEVER** for backrest release.
5. Proceed with facility protocols.

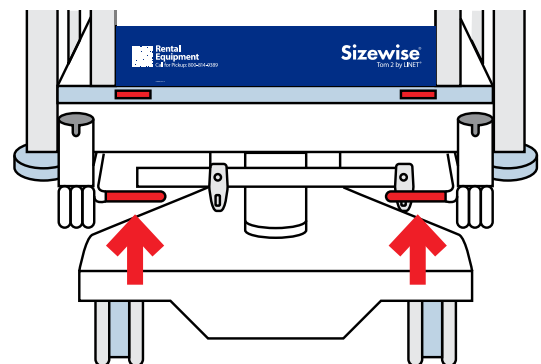

CPR Lever
Head of bed