

# Pediatric Pulse™ Instructions

sizewise.com 800-814-9389

**Lockout**  
Lockout will activate after three minutes of inactivity. To access control panel, press and hold for three seconds.

**Alarms**  
**Low Pressure Alarm**  
Press silence, then check power supply and all connections.  
**Power Failure Alarm**  
Press silence and check power supply.

**Firm and Soft**  
To increase air flow: use Firm and Soft up and down arrows.

**Autofirm**  
Press once for maximum inflation and a second time to remove. Autofirm will time out after 10 minutes and return to previous settings.

**Mode**  
Press Mode button once for Static therapy and a second time for Pulsation. Adjust cycle times from 3-20 minutes using up and down arrows.

**Fowler**  
Fowler feature will automatically start when head section is raised.

**QuickReference**

|  |   |   |
|--|---|---|
| <b>Lockout</b><br>Press and hold for 3 seconds to access control panel.                  | <b>Firm</b><br>Increases air flow for a 30 sec setting.                         | <b>Autofirm</b><br>Pushes the available pressure to maximum frequency. Time out setting at 10 min. Returns to previous setting. |
| <b>Low Pressure Alarm</b><br>Press silence, then check power supply and all connections. | <b>Soft</b><br>Decreases air flow for a 30 sec setting.                         | <b>Alarm Silence</b><br>Mutes the audible alarm. Press silence and the unit will auto lockout if required.                      |
| <b>Power Failure Alarm</b><br>Press silence and check power supply.                      | <b>Cycle Time</b><br>Increases or decreases the time of cycle between 3-20 min. | <b>Lockout</b><br>Lockout mode of machine automatically after 3 min. To activate press and hold lockout button for 3 sec.       |

**Sizewise**

## FOR CPR

- Turn off power switches on control unit
- Twist and open red CPR valve

## FOR TRANSPORT

- Turn off power switches on control unit
- Unplug power cord from wall