



Platinum 6000™

QUICK REFERENCE GUIDE



KEYPAD CONTROLS



Power/Standby
Green LED: Power On
Amber LED: Standby



Immersion with microclimate



Lock/Unlock




Alternation with microclimate



Mode



Autofirm

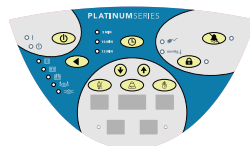

Power Fail



Immersion



Cycle Time
Alternation Therapy

mmHg ↓ Low Pressure



Alternation



Alarm Silence

OPERATING INSTRUCTIONS

1. Press and hold . Green LED indicates control unit is on; Amber LED indicates standby.
2. Press until is selected to autofirm surface; transfer patient to surface.

SETTING THERAPY MODE

1. Use to select Therapy Mode:



Alternation



Immersion



Alternation with microclimate



Immersion with microclimate

2. Alternation mode requires a 5, 10 or 15 min. cycle. Use to adjust.

PATIENT-SPECIFIC ADJUSTMENTS

To customize the mattress for your patient, follow these three steps:



Patient Height:

Select and use until patient's height is shown on display.



Patient Weight:

Select and use until patient's weight is shown on display.



Comfort Settings:






Select and use to override auto pressure settings and adjust firmness (15 to 50 mmHg).








CHANGING UNITS OF MEASUREMENT

The default setting is standard units (lb.). Decimal points in Patient Height or Patient Weight indicate metric units.

Change to Metric Units:

1. Put the control unit in standby by pressing .
2. Press and hold  and . Hold until all lights flash and you hear six beeps.
3. Cancel alarm by pressing .
4. Press  to turn on.

Change to Standard Units:


1. Put the control unit in standby by pressing .
2. Press and hold  and . Keep buttons depressed until all lights flash and you hear six beeps.
3. Cancel alarm by pressing .
4. Press  to turn on.

CPR/QUICK DEFLATION

1. Locate red CPR valve at head of mattress. Turn valve for quick air release.
2. Detach hoses from control unit by pressing blue button and pulling connector from unit.

PATIENT TRANSPORT

To maintain inflation during transport:

1. Ensure CPR valve is in closed position.
2. Press  to put control unit in standby mode (amber LED).
3. If mattress is to be disconnected from control unit, place transport cap over hose connections.