



# Platinum 6000™ (Rev 2)

QUICK REFERENCE GUIDE



## KEYPAD CONTROLS



**Power/Standby:**  
Green LED: Power On  
Amber LED: Standby



**Cycle Time:**  
For Alternation  
Therapy only



**Lock/Unlock**



**Alarm Silence:**  
Press to reset alarms



**Mode:**  
Select Immersion or  
Alternation



**Microclimate:**  
On by default; toggle  
to turn off




**Immersion**



**Autofirm:**  
Max inflate for  
transferring patient



**Alternation**

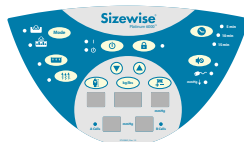

**Power Fail  
Warning**



**Low Pressure  
Warning**

## OPERATING INSTRUCTIONS

1. Press and hold . Green LED indicates control unit is on; Amber LED indicates standby mode.
2. Surface should automatically enter Autofirm mode. If not, press to max inflate surface.
3. Transfer patient to surface.

## SETTING THERAPY MODE

1. Use to select Immersion or Alternation mode.  
– Alternation requires a 5, 10 or 15 min. cycle. Use to adjust.
2. Microclimate is on by default. To toggle off, press .

## PATIENT-SPECIFIC ADJUSTMENTS

After setting Therapy Mode, customize the mattress for your patient by following three steps:



**Patient Height:**

Select and use until patient's height is shown on display.



**Patient Weight:**

Select and use until patient's weight is shown on display.



**Comfort Settings:**





Select to override auto pressure settings.  
Use to adjust firmness (15 to 50 mmHg).



## CHANGING UNITS OF MEASUREMENT

The default setting is standard units (lb.). Decimal points in Patient Height or Patient Weight indicate metric units.

### Change from Standard to Metric (or back):

1. Verify control unit is in standby or press  for two seconds to place into standby. Amber LED will show.
2. Press and hold  and  for five seconds.
3. Alarm will sound.
4. Press  to turn on.

---


## CPR/QUICK DEFLATION

1. Locate red CPR valve at head of mattress. Turn valve for quick air release.
2. Detach hoses from control unit by pressing blue button and pulling connector from unit.

---

## PATIENT TRANSPORT

To maintain inflation during transport:

1. Ensure CPR valve is in closed position.
2. Press  to put control unit in standby mode (amber LED).
3. If mattress is to be disconnected from control unit, place transport cap over hose connections.