



OPERATING INSTRUCTIONS

1. Locate both power switches on left side of control unit.
2. Turn both switches to **ON** position.



Mattress Inflation/Autofirm

1. Press **AUTOFIRM** button to fully inflate surface.
2. Transfer patient to surface. **AUTOFIRM** times out after 10 minutes and resets to previous setting. To manually exit, press **AUTOFIRM** a second time.



Pulsation Therapy

1. Set firmness using **FIRM/SOFT** buttons based on patient comfort and clinical discretion.
2. Press **PULSATE** button.
3. Press **PULSATE** button until desired time is reached (3, 5 or 10 min).



Immersion Therapy (Static)

1. Press **STATIC** button.
2. Set firmness using **FIRM/SOFT** buttons based on patient comfort and clinical discretion.

OTHER FEATURES



Fowler

Increases airflow when head section of bed is elevated. Newer models include an Auto Fowler feature.



Alarm Reset

Light remains illuminated until problem is corrected.



Lockout

Locks control panel (autolocks after 3 min). To unlock, press and hold for 3 sec.



Firm/Soft

Increases (**FIRM**) or decreases airflow (**SOFT**) to create a firmer or softer surface.

PATIENT TRANSPORT


1. Activate Autofirm mode for 3-5 minutes to ensure mattress is completely inflated.
2. Turn both power switches off (left side of control unit) and unplug from wall.
3. Mattress will slowly deflate. As patient sinks into mattress, patient will rest on foam transport pad.

CPR/QUICK DEFLATION

1. **CPR control is on the front of control unit.**

Depending on model, push red button or turn dial clockwise to activate CPR. Alarm will sound.

2. Locate hose connection to control unit (all models). Disconnect for quick air release.

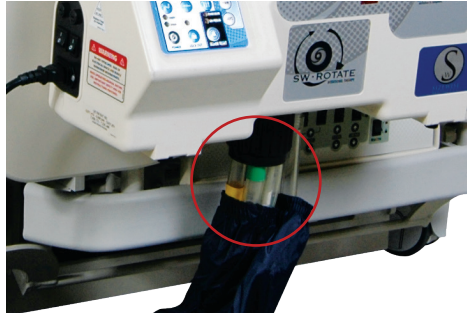

This guide is not meant to replace the product user manual.
For additional product support, call **800-814-9389**.