



Shuttle™ A and B Series

QUICK REFERENCE GUIDE



Hand Control

1. Stretcher/Table Position
2. Chair Position
3. Elevate Up/Down
4. Trendelenburg
5. Recline
6. Reverse Trendelenburg
7. Seat Assist Position



Leg Support Adjustment

To adjust leg support angle: pull lever, adjust angle. Release once angle is achieved.

To extend/retract footplate: push pin while holding onto footplate. Release once desired placement is achieved.



Armrest Adjustment

Loosen knob and raise/lower armrest to desired height; re-tighten knob.

Wheel Lock

- Steer:** Green up
- Neutral:** Middle
- Brake:** Red Down



SHUTTLE POSITIONS

Chair



1. Lock wheels.
2. Secure patient with seatbelt/harness.
3. Press the Chair button on the hand control.

Additional Adjustments

Height: Use Elevate Up/Down buttons to raise or lower Shuttle.

Recline: Press Recline button to tilt chair back.

Stretcher/Table



1. Lock wheels.
2. Ensure head of Shuttle and head of other device are aligned.
3. Secure patient with seatbelt/harness.
4. Press the Stretcher/Table button on hand control to flatten Shuttle.
5. Move the push bar into stretcher position by pushing button to release latch. Flip bar until it locks in place.

Note: NEVER sit on head section or leg supports when in Stretcher/Table position.

Additional Adjustments

Height: Use Elevate Up/Down buttons to raise or lower Shuttle.

Trendelenburg: Flip foot plates and push bar up. Press Trend button to place Shuttle in Trendelenburg position.

Reverse Trendelenburg: Flip foot plates and push bar up. Press Reverse button to place Shuttle in Reverse Trendelenburg position.

Seat Assist



1. Lock wheels.
2. Ensure stand assist poles are in place on the front of Shuttle.
3. Put Shuttle in Chair position.
4. Press the Seat Assist button to help tilt the chair forward.

Transfers

Refer to full User Manual for proper use of Shuttle in seated or lateral transfers.



Charging Shuttle

Plug Shuttle into an AC power source to charge. A full charge may take up to four hours. **Do not exceed six hours of charge time.**

Shuttle battery gauge is located on the left side of the base. A separate Power Drive battery gauge is located on the Power Drive control.



Push Bar

Move push bar between chair and stretcher position by pushing button to release latch.



Flip bar until it locks in place.

POWER DRIVE 360™ (some models)

Power Drive Layout

- A. Speed Control Dial
- B. Battery Gauge
- C. Forward/Reverse Switch
- D. On/Off Switch
- E. Controller Adjustment Knob
- F. Throttle

Using Power Drive

1. Unplug charger cord from the wall.
2. Put wheel locks in Neutral.
3. Pull out Controller Adjustment Knob (E) to adjust controller to desired position.
4. Use On/Off Switch (D) to turn power On. Wheel will engage with floor.
5. Set Speed Control Dial (A) to the slowest position.
6. Ensure Forward/Reverse Switch (C) is in Forward/FWD position.
7. Slowly squeeze Throttle (F). The more the throttle is squeezed, the faster Shuttle will go.
8. To steer, push controls in the opposite direction you want to go (ie: to turn left, push the controls to the right—and vice versa).
9. To stop, release Throttle.

Note: Shuttle is not recommended for use on a grade more than 5 degrees.



Emergency Release

In case of power failure, drive wheel will stay in contact with floor. To release drive wheel, press Emergency Release button, which can be found under the chair on the right side.

This guide is not meant to replace the product user manual. For additional product support, call **800-814-9389**.

OPS-0030-1023 Combines legacy QRGs 27650037 and 27650006. See user manual for full content.

