



1. All Locked
2. Head Up
3. Head Down
4. Knee/Foot Up
5. Knee/Foot Down
6. Cardiac Chair Up
7. Cardiac Chair Down
8. Key (Unlock)
9. Hi-Lo Lock
10. Bed Up
11. Bed Down
12. Trend
13. Reverse Trend
14. Under-Bed Light



To Lock Entire Bed

- Press the All Locked button **1**.
- Orange LEDs will appear across all functions.

To Unlock Entire Bed

- Press the All Locked button **1** and Key Button **8** at the same time.
- Orange LEDs will disappear across all functions.

To Lock a Single Function

- Press the function's top button **2**, **4**, **6**, **10** or **12** and Key Button **8** at the same time.
- Orange LEDs will appear when the function is locked.

To Unlock a Single Function

- Press the function's top button **2**, **4**, **6**, **10** or **12** and Key Button **8** at the same time.
- Orange LEDs will disappear when the function is locked.

To Lock Hi-Lo and Trend/Rev Trend

- Press the Hi-Lo Lock button **9**.
- Orange LED will appear when function is locked.

To Unlock Hi-Lo and Trend/Rev Trend

- Press the Hi-Lo Lock button **9** and Key button **8** at the same time.
- Orange LED will disappear when function is unlocked.