


# ADAPT PRO ELITE™

## QUICK REFERENCE GUIDE

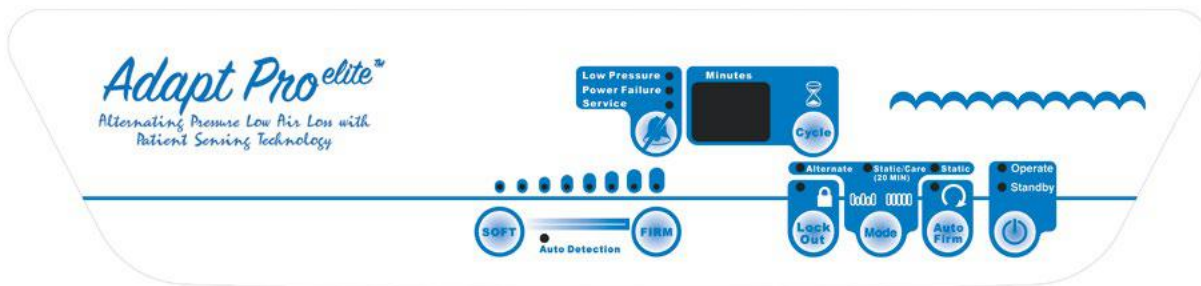

Alarm Mute and Alarm Indicator



Alternate Cycle Time or Warning code Display



Operating or Standby



Auto-Firm



Function Mode Selection (Alternate/Static Care/Static)



Panel Lock-out



CT Comfort Control



Auto Detection

Please see operator's manual for further information at [www.bluechipmedical.com](http://www.bluechipmedical.com).  
For further questions, please call your local Agiliti office or call 1.800.847.7368.

# ADAPT PRO ELITE™

## QUICK REFERENCE GUIDE



### Product Operation Guide



1. Press **power** button on side of pump to turn on the unit. LED indicators on the control panel will light up accompanied with a beep for 2 seconds.



2. Press the **Operate** button and the system will begin to inflate and the "Auto-Firm" indicator will be flashing.



3. After initial inflation is completed. Press **Auto-Firm** button for moving the patient onto the mattress. The mattress will be in a static mode in approximately 5 minutes.



4. Activate **Auto Detection** by pressing the **Soft** and **Firm** buttons together until LED blinks, and will automatically choose the comfort level setting based on weight. Comfort level setting can be adjusted manually by pressing either Soft or Firm buttons



5. Press **Mode** button to choose Alternate, Static Care or Static Therapy modes



6. Press **Cycle** button to choose Alternate cycle time between 10-30 minutes



7. Press **Lock Out** button to lock in therapy mode. To release hold button until LED is off



8. For Patient Transport, press **Auto Firm** button, wait for full inflation, disconnect hose from pump and put on hose connector transport cap to keep mattress inflated.



9. **CPR** - Twist CPR and Bolster Release Valve in either direction to release mattress air.