




Siderail Removal/Installation

Pull knob and lift rail or lower into mount.

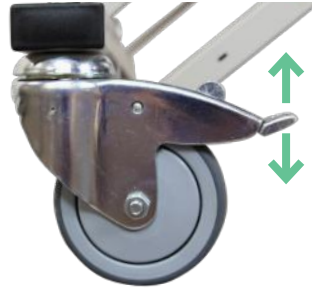

Casters

- To Lock, press lever down
- To Unlock, raise lever




Expanding/Retracting Deck (some models)

Lift handle to unlock while pulling out or pushing in section.

Expand/retract all sections on both sides for 39"/42" and 48" widths.

Hand Control

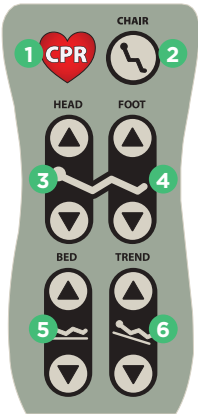


1. Hold to flatten bed for CPR
2. Cardiac Chair position
3. Head section
4. Foot section
5. Bed height
6. Trend/Reverse Trend
7. Control Buttons
8. Lockout Switches
9. Master Lockout Switch

Footboard Controls

Located at foot end of bed. Controls include individual lockout functions as well as master lockout switch.

Locking out specific or all functions will disable affected functions on both hand and foot control.

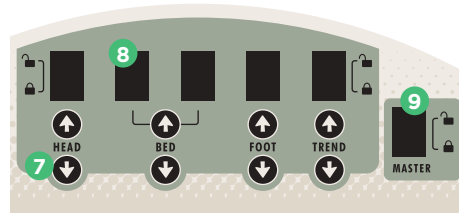

CODE SITUATION

1. Unlock casters and pull bed from wall.
2. Engage brakes, remove headboard, and remove or lower siderails for patient access.
3. For Air Support Surfaces: press CPR button or turn CPR dial on control unit and/or open CPR valve at head of mattress if installed.
4. **All beds:** press and hold CPR button on hand control to flatten bed.
Some models: pull and hold CPR handle located on head section of deck.
5. Proceed with facility protocols for resuscitation.

Troubleshooting

If bed fails to function:

1. Verify all functions on footboard control panel are unlocked
2. Verify master switch is unlocked
3. Verify power cord is plugged into working AC outlet and not cut, frayed or damaged
4. Inspect patient hand control for damage and ensure it is properly connected

If individual function does not operate:

1. Verify all functions on footboard control panel are unlocked
2. Verify master switch is unlocked

If bed will not operate on battery power:

Verify patient hand control is properly connected

