


Siderail Adjustment

To lower siderail: push button and rotate away from center of bed.

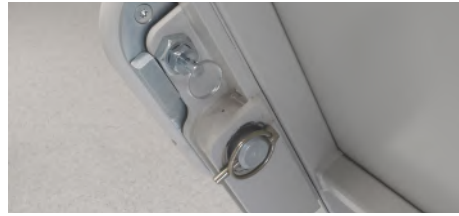

Siderail Removal

To remove: Pull pin and slide siderail away from frame.



Expanding/Retracting Deck

Position knob to UNLOCK while pulling out section.

Expand all sections on both sides to 48" width.

Hand Control



1. Hold to flatten bed for CPR
2. Cardiac Chair position
3. Head/Foot section
4. Bed height
5. Trend/Reverse Trend

In Case of Power Outage



Emergency Hand Crank may be used to articulate bed.

Casters

- To Lock, press lever down
- To Unlock, raise lever



Troubleshooting

If bed fails to function:

1. Verify hand control is plugged into control module at head of bed.
2. Check that both power switches are turned on under head of bed.
3. Check power cord is properly plugged in.
4. Change power source

SCALE METER


System Setup

1. For most accurate weight measurement, casters should be parallel to bed frame and pointing toward center of bed.
2. Bed cannot be in Trend or Reverse Trend position.
3. Scale Meter should remain in holder when weighing patient.
4. Do not lean on bed when weighing patient.
5. For beds with Power Drive, see back of card.

Weighing Patient

Press **ON/WEIGHT** and view patient weight on display.

System will power down after 60 seconds of inactivity. Weight readings are not saved.

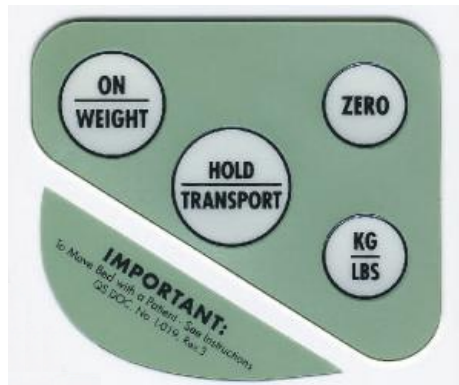

Zeroing the Scale Meter

NOTE: Never zero Scale Meter with patient in bed.

NOTE: Scale Meter should not be zeroed while in kg mode.

1. Press and hold **ON/WEIGHT** and **ZERO** until the message "Recalibrating Zero Scale: -00-" appears.
2. Release buttons.

To Change Unit of Measure

Current unit of measure (kg or lbs.) appears in bottom right of display.

To change, press **KG/LBS**.

Adding/Removing Items

Press **HOLD/TRANSPORT** when adding or removing items from bed. This feature must also be used when moving bed.

Transport

Before Transport: Press **ON/WEIGHT** to view patient weight on display. Then press **HOLD/TRANSPORT** to preserve original weight reading. Adjustments may now be made to bed or patient.

After Transport: Press **HOLD/TRANSPORT** to return to original weight reading.

Then press **ON/WEIGHT**.

CAUTION: if left in **HOLD/TRANSPORT**, Scale Meter will not power down and batteries may drain.