

SENTRY 1400

LOW AIR-LOSS & ALTERNATING PRESSURE MATTRESS
FOR PATIENTS UP TO 500 LBS (226 KG)



GETTING STARTED



1. Press POWER ON/OFF button on the top right corner of the controller screen



2. Press MAX for rapid inflation

3. To allow low air-loss (LAL) flow under patient, use dry flow chux and/or draw sheet as appropriate

4. Zero the hospital bed frame scales

5. Place patient in center of mattress

6. Obtain weight from bed frame

7. To program patient's weight:




- Press menu

- Use the Up/down arrows to select patient's weight (lbs/kgs)

- Press menu to save settings



8. Press Therapy, choose between Alternating Pressure (AP) or Floatation (FLT)

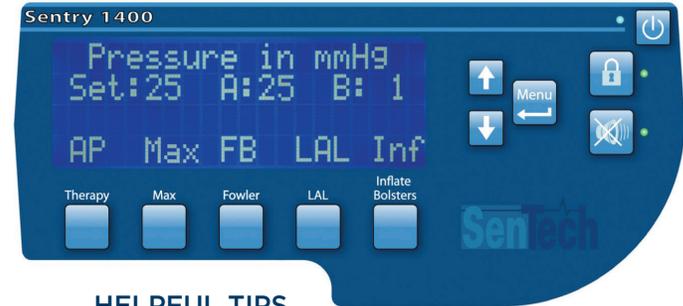

HELPFUL TIPS



- Select **Fowler Boost (FB)** when patient's head of bed (HOB) is elevated; turn off when HOB is flat



- To ensure **LAL** remains on at all times periodically check the controller



- Press **Inflate bolsters (INF)** for patients at risk of falls

- Deflate bolsters when patient is entering or exiting the bed

- For most patients optimal therapy is **AP**

SENTRY 1400

LOW AIR-LOSS & ALTERNATING PRESSURE MATTRESS
FOR PATIENTS UP TO 500 LBS (226 KG)



DEFINITION



Therapy

- **Alternating Pressure (AP):** Alternates pressure of every other baffle (Zone A and Zone B).
- **Flotation (FLT):** Equalizes pressure in all baffles.



Max

When activated the box will display MAX. This inflates the entire bed to maximum pressure for 20 minutes. After, it will revert back to the previous setting.



Fowler Boost (FB)

Press when patient is sitting to add an additional 20% of air into the center of the mattress. Turn off when patient is lowered back to flat/supine position. You can adjust these settings in the Advanced Settings menu.



Low Air-Loss (LAL)

Provides gentle air flow to skin to help keep patients dry. When the LAL setting is activated, it will slowly diffuse air through the coverlet.



Inflate Bolsters (INF)

Activates and deactivates the bolsters. They will take approximately one minute to inflate.



MENU/Advanced Settings

Use to customize mattress pressures, percentage of FB and AP cycle times. From the main Menu select Advanced Settings, make desired changes and press MENU to save settings.



Lock

Press and hold for two seconds to activate/deactivate - locks in settings to prevent unintended changes.



Alarm Mute

Press to deactivate alarm and check all connections and tubes. If alarm continues, please contact Agiliti.

ADDITIONAL INFORMATION

CPR

The CPR valve is on the lower left side of the control box. Align the CPR pull key towards the outside so it is visible and easily accessible. To release the CPR valve, press and squeeze the tabs while pulling down. The mattress will rapidly deflate to the foam surface to perform CPR.

Power Loss

All settings are saved when power returns (with exception of ALARM MUTE and LOCK functions).

Cleaning

Coverlet can be wiped with standard cleaning wipes. If heavily soiled, contact Agiliti for replacement.

Please see operator's manual for further information at www.tridien.com.
For further questions, contact your local Agiliti office or call **1.800.847.7368**.