
Important Before You Start: THIS IS AN ACCESSORY “QUICK SET-UP” GUIDE ONLY

Please refer to the *Dabir Operation / Service Technical Manual* for full System set-up and usage instructions. Failure to carefully read and follow all instructions and safety information may result in decreased product performance or injury.


Battery Insertion & Charging Instructions:

1. Remove battery from packaging. (**NOTE: INITIAL CHARGING WILL TAKE UP TO 2 HOURS.**)
2. Insert bottom of battery into the controller pocket, then pivot inwards until an audible “CLICK” occurs. It is important to note that the battery can be installed while the controller is either ON or OFF. The controller does not need to be ON to charge the battery, but it must be plugged in.
3. When properly connected, the battery icon will appear. If the icon is not present, battery is NOT properly connected and Step 2 MUST be repeated.

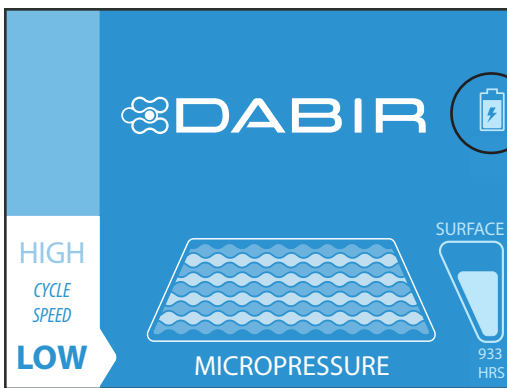Step 3: Battery Icon Check

- Connected & Charging
- Charge Level
- = Low Charge (Yellow)
- = Battery Defective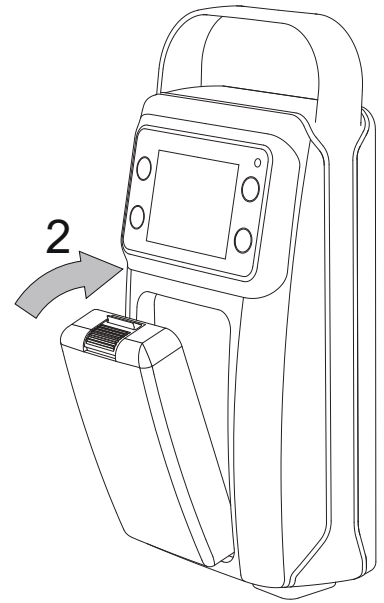

NOTE: The Battery will automatically charge when AC power is supplied to Controller.


Battery Removal:

NOTE: After battery removal, controller MUST be connected to AC power source to maintain session.

4. To remove battery from controller, grip the battery by its sides and depress the orange button.
5. Rotate battery outward and remove as shown.

Checking Battery Charge Level:

6. Press “PUSH TO TEST”, to activate charge level LED indicator.
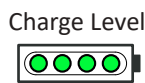


NOTE:

It is the responsibility of the user to properly clean and disinfect the System prior to patient use.

Cleaning Procedure: www.dabir-surfaces.com/IFU

Approved Cleaners: www.dabir-surfaces.com/cleaners

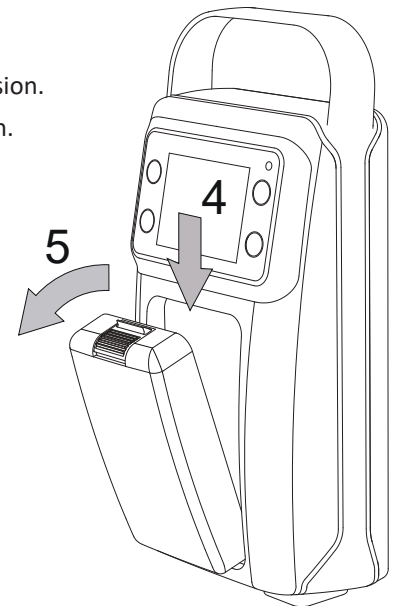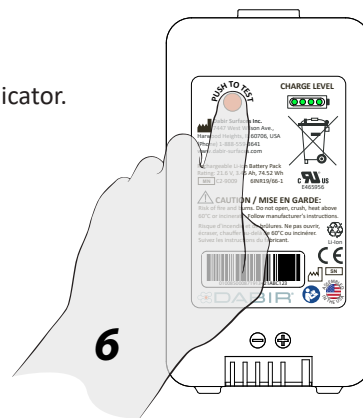Corporate Headquarters, Sales & Manufacturing:

Dabir Surfaces, Inc.

Sales Inquiries:

Technical Support:

Authorized European Representative:

PATENT:

Distributor (North America): STERIS Corporation, Customer Service, Tel: +1(800)548-4873, Fax: +1(440)639-4450; Service Support, Tel: +1(800)333-8828

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Dabir Surfaces, Inc. 7447 West Wilson Ave., Harwood Heights, IL 60706, États-Unis, www.dabir-surfaces.com