

NP Adjust™



User Manual

Doc. #: _____ Rev. Date: _____ Rev.: _____

Table of Contents

- Introduction..... 3
 - Product Description 3
- Product at a Glance..... 4
- Symbols 5
- Labels..... 6
- Safety Instructions (cont.)..... 8
 - Seven Zones of Bed Rail Entrapment.... 8
- Specifications 11
- Unpacking Instructions 13
- Set-Up Instructions 13
- Cleaning Instructions 15
- Preventive Maintenance 17
- Storage, Transport and Disposal 18
- Limited Product Warranty 19

Introduction

Product Description

The NP Adjust™ non-powered support surface can accommodate diverse patient body types and patient weights. The system consists of a non-skid mattress base, a series of foam-filled air cells encased in a foam cavity and a low friction, anti-shear top cover. The surface and components are designed to eliminate harmful chemicals of concern in the fabric, fire barrier and foam. A layer of CuPro™ copper- and gel-infused foam is laminated to the top of the foam cavity and serves as the support surface underneath the patient. The foam cavity also includes bolsters on the sides and ends of the support surface, providing added patient stability and positioning. The sloped heel section aids in patient comfort and pressure injury prevention.

Intended Use

The product is a non-powered, zoned pressure redistribution support surface intended to prevent and treat pressure injuries. The product is for use on a medical bed frame in a medical facility by qualified personnel.

General Information

- Read and completely understand the manual before use.
- DO NOT modify this equipment without authorization from the manufacturer.
- Only use Agiliti replacement covers that are specific to this product.
- At certain intervals, maintenance of this product is required and must be performed by qualified personnel only.
- Retain this manual for future use and maintenance.
- Users should be risk assessed to ensure they are able to understand this manual and use this product without risk to themselves or others.

Symbols, definitions and fonts are used throughout this manual to aid user readability and understanding of content.






- Standard Text: Used for regular information.
- **Bold Text**: Emphasizes a word or phrase.
- **NOTE**: Sets apart special information or important instruction clarification.

Product at a Glance

NP Adjust™



Symbols


Symbol	Description
	<p>Follow User Manual: Indicates mandatory action to read and fully understand these instructions prior to use.</p>
	<p>Accompanying Documents: Read and fully understand all accompanying documents for product before use.</p>
	<p>ELECTRICAL SHOCK HAZARD WARNING: This symbol is intended to alert the user to the presence of electrical shock hazards. It is important to follow all instructions and special procedures to avoid electrical shock to the operator, care provider, and/or patient.</p>
	<p>WARNING: Indicates a potentially dangerous situation, which if not avoided, could result in injury ranging from moderate to serious severity up to and including death.</p>
	<p>CAUTION: Indicates a potentially dangerous situation, which if not avoided, could result in injury ranging from minor to moderate severity.</p>

Labels


Label	Description
-------	-------------




Screen printing on mattress identifies support surface name/version.

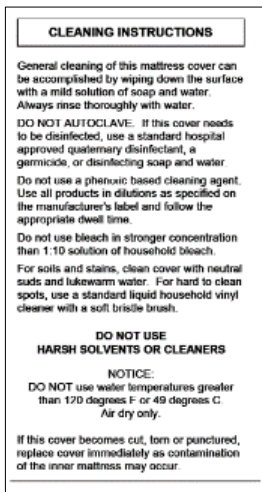

Label provides support surface cleaning instructions.

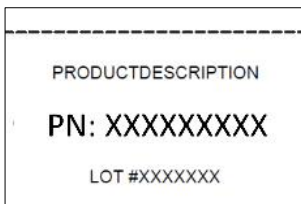

Labels identify support surface top cover/bottom cover part and lot numbers.

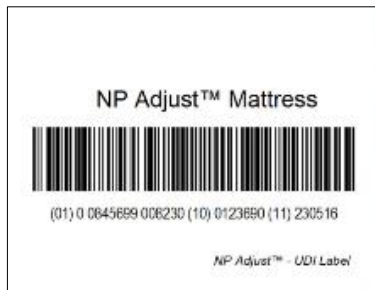

Label provides the Unique Device Identification (UDI) number of support surface.

<p>UNDER PENALTY OF LAW THIS TAG IS NOT TO BE REMOVED EXCEPT BY THE CONSUMER</p>
<p>ALL NEW MATERIAL CONSISTING OF POLYURETHANE FOAM 100%</p>
<p>Certification is made by the manufacturer that the materials in this article are described in accordance with law.</p>
<p>REG. NO. CA 46736</p>
<p>MADE FOR Agility Health, Inc. 11065 Viking Dr., Ste 300 Eden Prairie, MN 55344</p>
<p>Finished Size: Net Weight of Filling Mat: 23 lbs. Oz: Mattress</p>
<p>MADE IN THE USA</p>
<p>MANUFACTURED IN: MONTH _____ YEAR _____</p>

Label identifies legal details of support surface manufacturing.

<p>Manufactured by: Sewee Manufacturing 232 Marsh Circle Corona, CA 92679-1751 Date of Manufacture: Model : Prototype ID: This mattress meets the requirements of 16 CFR Part 1633 (federal flammability (open flame) standard for mattresses sold) when used without a foundation.</p>
<p>THIS MATTRESS IS INTENDED TO BE USED WITHOUT A FOUNDATION</p>

Label identifies support surface compliance with 16 CFR Part 1633.

<p>NOTICE</p> <p>THIS ARTICLE MEETS THE FLAMMABILITY REQUIREMENTS OF CALIFORNIA BUREAU OF HOUSEHOLD GOODS AND SERVICES TECHNICAL BULLETIN 117-2013. CARE SHOULD BE EXERCISED NEAR OPEN FLAME OR WITH BURNING CIGARETTES.</p> <p>THE UPHOLSTERY MATERIALS IN THIS PRODUCT:</p> <p>_____ CONTAIN ADDED FLAME RETARDANT CHEMICALS</p> <p>X CONTAIN NO ADDED FLAME RETARDANT CHEMICALS</p> <p>THE STATE OF CALIFORNIA HAS UPDATED THE FLAMMABILITY STANDARD AND DETERMINED THAT THE FIRE SAFETY REQUIREMENTS FOR THIS PRODUCT CAN BE MET WITHOUT ADDING FLAME RETARDANT CHEMICALS. THE STATE HAS IDENTIFIED MANY FLAME RETARDANT CHEMICALS AS BEING KNOWN TO, OR STRONGLY SUSPECTED OF, ADVERSELY IMPACTING HUMAN HEALTH OR DEVELOPMENT.</p> <p>TB117-2013</p>
--

Label identifies support surface compliance with Technical Bulletin 117-2013.

Safety Instructions





WARNING: Bed rail entrapment is a serious health risk that can result in serious injury or even death. Agiliti recommends the care provider be mindful of the FDA guidelines relevant to bed rail entrapment, as serious injury can result. These guidelines can be found on the FDA website. When using a foam or air therapy system the care provider is responsible for ensuring the support surface properly fits the bed frame. It is also the care provider's ultimate decision whether to use bed rails with the patient.

Seven Zones of Bed Rail Entrapment

If the patient has any clinical conditions that could result in risk of falling or improperly lying-in bed, the bed should be left at its lowest setting and in flat position when not attended. Agiliti recommends the use of bed rails if they are available. There are seven zones of bed rail entrapment.




Zone 1:
Within the Rail

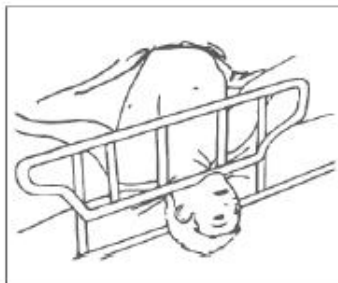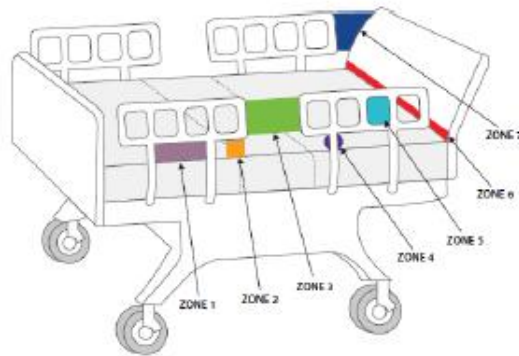


Zone 2:
Under the Rail, Between the Rail Supports, or Next to a Single Rail Support



Zone 3:
Between the Rail and the Mattress

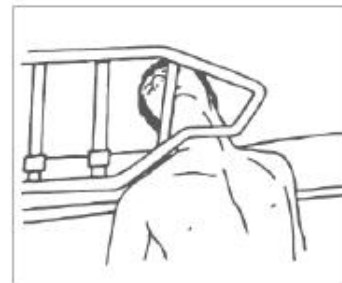


Zone 4:
Under the Rail, at the Ends of the Rail

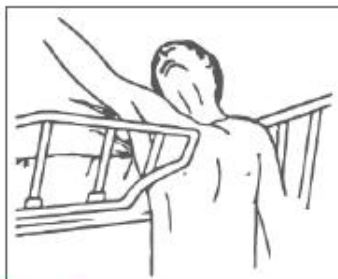

Zone 5:
Between Split Bed Rails



Zone 6:
Between the End of the Rail and the Side Edge of the Head or Footboard

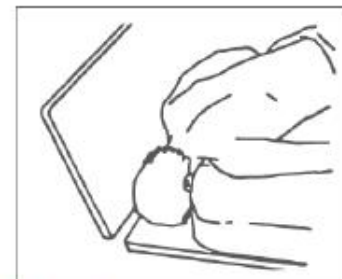

Zone 7:
Between the Head or Footboard and the Mattress End

Reference: Hospital Bed System Dimensional and Assessment Guidance to Reduce Entrapment. (2006, Mar. 10). U.S. Health and Human Services Food and Drug Administration Center for Devices and Radiological Health. Retrieved from <https://www.fda.gov/downloads/medicaldevices/deviceregulationandguidance/guidancedocuments/ucm072729.pdf>

Safety Instructions (cont.)

Users of the NP Adjust™ support surface should ensure a thorough understanding of the bed frame user manual prior to installation and use. Follow the instructions contained in this manual carefully. Use NP Adjust only if it is in proper working order and free from tears or punctures. The terms below are used frequently throughout this manual and must be understood by all users.



ELECTRICAL SHOCK HAZARD WARNING: This symbol is intended to alert the user to the presence of electrical shock hazards. It is important to follow all instructions and special procedures to avoid electrical shock to the operator, care provider and/or patient.



WARNING: Indicates a potentially dangerous situation, which if not avoided, could result in injury ranging from moderate to serious severity up to and including death.



CAUTION: Indicates a potentially dangerous situation, which if not avoided, could result in injury ranging from minor to moderate severity.

Electrical Shock Hazards



ELECTRICAL SHOCK HAZARD WARNING: To avoid risk of electric shock or injury, ensure all cords on bed frame are away from bed deck prior to installing product.



ELECTRICAL SHOCK HAZARD WARNING: Before any cleaning and disinfection procedures are performed, unplug all power cords in proximity to product to eliminate possibility of electrical shock.

Warnings



WARNING: To avoid serious injury or death, ensure product is used with compatible bed frame sizes.



WARNING: To avoid serious injury or death, ensure product is properly retained on bed frame to avoid slippage or unintended movement.



WARNING: Product is to be used in accordance with each facility's policies and procedures.



WARNING: Do not exceed Max. Patient Weight listed in the specifications. Doing so can lead to injury or damage



WARNING: Bed rail entrapment can result in serious injury or even death. Agiliti recommends caregivers be mindful of the FDA guidelines relevant to bed rail entrapment. These guidelines can be found on the FDA website. When using a replacement support surface, the caregiver is responsible for ensuring the product properly fits the bed frame. It is also the caregiver's decision whether to use bed rails with the patient.

Cautions



CAUTION: Use only approved replacement parts or covers.



CAUTION: Use of product is only one element of care in the prevention and treatment of pressure injuries. Frequent repositioning, proper care, routine skin assessment, wound treatment and proper nutrition are but a few of the elements required in the prevention and treatment of pressure injuries. The responsibility of prevention and treatment of pressure injuries lies with the health care professional.

Safety Instructions (cont.)



CAUTION: Ensure that product covers are free from tears, rips or damage from shipping and handling prior to installing on bed frame.



CAUTION: It is recommended that gloves and protective clothing be worn during cleaning and disinfecting.



CAUTION: Product requires regular maintenance to ensure performance and avoid premature wear, damage, and injury.

General Notes

NOTE: Only the outer cover requires cleaning and maintenance. Disassembly of the support surface for maintenance of internal components is not recommended. Do not use harsh solvents or cleaners. Use care to avoid puncturing the product with needles or sharp instruments. This may result in loss of integrity of the cover or internal components.

NOTE: To ensure that approved chemical agents are being used for the top and bottom covers, visit agilityhealth.com/approvedsurfacecleaners.

Specifications

NP Adjust™ Support Surface		
Width	35"	89 cm
Length (standard)	82"	208 cm
Length (extendable)	82" to 88"	208 cm to 224 cm
Height	7"	18 cm
Safe Working Load	550 lb.	249 kg
Max. Patient Weight*	500 lb.	227 kg
Support Surface Weight	34 lb.	15 kg
NP Adjust EX™ Support Surface (Seamless Expandable)		
Width	39" to 48"	99 cm to 122 cm
Length (standard)	82"	208 cm
Length (extendable)	82" to 88"	208 cm to 224 cm
Height	7"	18 cm
Safe Working Load (39" / 48")	750 lb. / 1,050 lb.	340 kg / 476 kg
Max. Patient Weight* (39" / 48")	700 lb. / 1,000 lb.	318 kg / 454 kg
Support Surface Weight	50 lb.	23 kg
NP Adjust BH™ Support Surface (Behavioral Health)		
Width**	35" or 39"	89 cm or 99 cm
Length (standard)	82"	208 cm
Height	7"	18 cm
Safe Working Load (35" / 39")	550 lb. / 750 lb.	249 kg / 340 kg
Max. Patient Weight* (35" / 39")	500 lb. / 700 lb.	227 kg / 318 kg
Support Surface Weight (35" / 39")	31 lb. / 32 lb.	14 kg / 15 kg

* NOTE: Max Patient Weight capacity of support surface is not to exceed bed frame Safe Working Load.

** NOTE: Support Surface available with Defined Perimeter

Specifications (cont.)

Components	Material	Features
Top Cover	Polycarbonate	4-way stretch, breathable antimicrobial
Bottom Cover	200D Laminated TPU	Hi-friction nonslip
Foam Cavity	Polyurethane foam cavity with CuPro™ Visco foam topper	Heel slope
Air Cells	Trilaminate polyurethane foam-filled air cells	3 zones of air cells to support body sections
Core Shield™	Stretch polyurethane film	Flexible foam core protection

Unpacking Instructions



CAUTION: Ensure that support surface covers are free from tears, rips or damage from shipping and handling prior to installing on bed frame.

1. Remove products from packing materials and examine for shipping damage. Notify Agiliti immediately upon discovery of damage.
 - a. *If delivered by a freight company:* note a description of the damage on the carrier's bill of lading and take photos for your records. Depending on the shipping arrangement, the obligation to prepare, file, and pursue any claim for any damage or loss may be on the shipping recipient (or buyer).
 - b. *If delivered by an Agiliti employee:* please reach out to your local contact to initiate the process.
2. If replacement parts are needed, contact Agiliti Customer Service.
3. Remove packaging surrounding product.

Set-Up Instructions



WARNING: Bed rail entrapment can result in serious injury or even death. Agiliti recommends caregivers be mindful of the FDA guidelines relevant to bed rail entrapment. These guidelines can be found on the FDA website. When using a replacement support surface, the caregiver is responsible for ensuring the product properly fits the bed frame. It is also the caregiver's decision whether to use bed rails with the patient.



CAUTION: Use of product is only one element of care in the prevention and treatment of pressure injuries. Frequent repositioning, proper care, routine skin assessment, wound treatment and proper nutrition are but a few of the elements required in the prevention and treatment of pressure injuries. The responsibility of prevention and treatment of pressure injuries lies with the health care professional.



CAUTION: Ensure that support surface covers are free from tears, rips or damage from shipping and handling prior to installing on bed frame.

NP Adjust™ and NP Adjust BH™

1. Verify that product is the correct size for bed frame top deck. Refer to *Seven Zones of Bed Rail Entrapment* guidelines (see Table of Contents).
2. Place bed frame in a flat position.
3. Install product with the sloped heel section toward the foot of the bed.
4. Disinfect product prior to patient use. Refer to *Cleaning Instructions* (see Table of Contents).

Set-Up Instructions (cont.)


NP Adjust EX™



WARNING: To avoid serious injury or death, ensure side bolsters are properly installed and secured to mattress to avoid slippage or unintended rotation.



WARNING: Bed rail entrapment can result in serious injury or even death. Agiliti recommends caregivers be mindful of the FDA guidelines relevant to bed rail entrapment. These guidelines can be found on the FDA website. When using a replacement support surface, the caregiver is responsible for ensuring the product properly fits the bed frame. It is also the caregiver's decision whether to use bed rails with the patient.

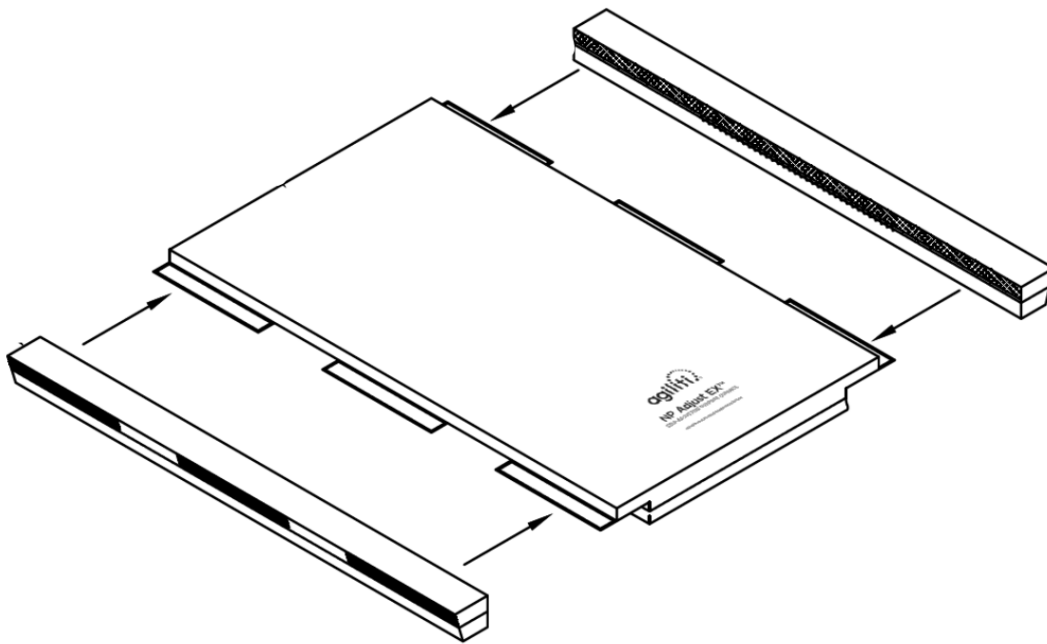

1. Verify that product is the correct size for bed frame top deck. Refer to *Seven Zones of Bed Rail Entrapment* (see Table of Contents).
2. Place bed frame in a flat position.
3. Install product with the NP Adjust EX™ logo facing up toward the foot of the bed.
4. Install side bolsters under the top foam flap with narrow width on bed deck surface.
5. Secure side bolsters to mattress by ensuring top foam flap overlaps side bolster and Velcro lines up.
6. Press on Velcro to secure final assembly.
7. Disinfect product prior to patient use. Refer to *Cleaning Instructions* (see Table of Contents).

Cleaning Instructions



ELECTRICAL SHOCK HAZARD WARNING: Before any cleaning and disinfection procedures are performed, unplug all power cords in proximity to product to eliminate possibility of electrical shock.



CAUTION: Gloves and protective clothing should be worn during cleaning and disinfecting.



CAUTION: Product requires regular maintenance to ensure performance and to avoid premature wear, damage, and injury.

NOTE: Only the outer covers of the product require cleaning and maintenance. Disassembly of the support surface for cleaning or maintenance of internal components is not recommended. Do not use unapproved cleaners. Use care to avoid puncturing the support surface with needles or sharp instruments. This may result in loss of integrity of the support surface cover or internal components.

NOTE: To ensure that approved chemical agents are being used for outer covers, visit agilityhealth.com/approvedsurfacecleaners.

DO NOT autoclave.

NOTE: Improper cleaning, rinsing or the incorrect use of cleaning agents can lead to premature fabric discoloration and breakdown of the fabric's fluid resistance, stain resistance and fabric strength.

Top Cover

The top cover material is resistant to a wide variety of hospital cleaners and resists breakdown with disinfectants and cleaners based on chlorine, alcohol, peroxide, quaternary ammonium, chloramine, and phenol. Personal Protective Equipment (PPE) should always be used as directed by the disinfectant's Material Safety Data Sheet.

1. Prepare the disinfectant according to the manufacturer's recommendations.
2. Prepare a separate bucket of warm, fresh water to be used for rinsing the equipment after cleaning/disinfecting procedures are completed as instructed.
3. All exterior areas of the top cover are to be wiped using a cloth dampened with the disinfectant, prepared as directed by the manufacturer's recommendations, to remove organic material and visible soil.
4. Allow the support surface shell to remain wet with disinfectant solution for the manufacturer's recommended contact time.
5. Immediately after recommended contact time, rinse all surfaces of product with fresh water and a clean cloth to remove chemical and organic residue.
6. Stains on the top cover may be treated using an approved stain remover.

NOTE: To ensure that approved chemical agents are being used for outer covers, visit agilityhealth.com/approvedsurfacecleaners.

- No abrasives: Abrasive cleaners and abrasive cleaning devices (scouring sponges) will dramatically reduce the life of the top cover.
- Do not use solvents.
- Follow dilution recommendations: One of the most significant errors in cleaning support surface top covers is not diluting a cleaner according to the manufacturer recommendations. Most surface cleaners are sold in concentrated forms and are intended to be diluted, usually with a large percentage of water. Failing to do this can shorten the life of the support surface top cover significantly.
- Rinse after using a cleaning agent: Leaving the cleaner to dry on the surface will shorten the lifespan of the top cover.

NOTE: If additional cleaning is necessary, the top cover may be removed and laundered using standard hospital disinfectant/detergent. DO NOT exceed 170°F (76°C).

1. Set washing machine to Regular Cycle.
2. Pre-soak with disinfectant/detergent in cold water for 10 minutes. DO NOT use unapproved cleaners.
3. Main wash cycle: 15 minutes (time dependent on soil level).

Cleaning Instructions (cont.)

4. Rinse cycle: 5 minutes, minimum.
5. Spin/Drain cycle: 5 minutes, minimum

NOTE: After washing, the top cover must be air dried or in a dryer at very low or no heat to protect it from heat-related damage.

Bottom Cover

1. Prepare the disinfectant according to the manufacturer's recommendations.
2. Prepare a separate bucket of warm, fresh water to be used for rinsing the equipment after cleaning/disinfecting procedures are completed as instructed.
3. All areas of the bottom cover are to be wiped using a coarse cloth dampened with the disinfectant, prepared as directed by the manufacturer's recommendations, to remove organic material and visible soil.
4. Allow the bottom cover to remain wet with disinfectant solution for the manufacturer's recommended contact time, and no longer.
5. Immediately after recommended contact time, rinse all areas of the bottom cover with fresh water and a clean cloth to remove chemical and organic residue.
6. Stains on the bottom cover may be treated using an approved stain remover, according to the chemical manufacturer's recommendations.
7. After washing, the bottom cover must be allowed to air dry.

Preventive Maintenance



CAUTION: Gloves and protective clothing should be worn during maintenance, cleaning, and disinfecting.



CAUTION: Preventive maintenance should not be attempted while equipment is in use.

NOTE: To keep equipment working efficiently and effectively for an extended time, it is essential to make sure the equipment is cleaned regularly and well maintained.

Check the items on this chart at the indicated intervals. Immediately replace any loose, worn, or damaged items. Place product on flat, clean, and level surface and inspect for damage, wear, punctures, rips and/or tears on the following components:

Components	Inspect for	During cleaning and disinfecting	Annually
Top Cover	Damage, wear, punctures, rips and/or tears	X	X
Bottom Cover	Damage, wear, punctures, rips and/or tears	X	X
Core Shield™	Damage, wear, punctures, rips and/or tears	X	X
Foam Cavity	Remove covers and inspect for fluid ingress	N/A	X

Storage, Transport and Disposal

- Always store product on a clean, level surface.
- Avoid storage of other equipment on top of product.
- Use care during transport to prevent damage.
- End-of-life Agiliti products must be disposed of properly according to local laws and regulations. Please contact Agiliti or your local authorities for disposal and recycling options.

Limited Product Warranty D-L00-001 Rev. 5

NP Adjust™

PLEASE READ THESE WARRANTY TERMS AND CONDITIONS CAREFULLY BEFORE USING YOUR AGILITI PRODUCT. BY USING THE PRODUCT, YOU ARE CONSENTING TO BE BOUND BY THE FOLLOWING WARRANTY TERMS AND CONDITIONS.

Limited Warranty

Raye's, Inc dba Sizewise Manufacturing ("Sizewise Manufacturing") warrants the above referenced product to be free from material defects in materials and workmanship for the warranty periods set forth below when the product is used and serviced properly in accordance with the specifications set forth in the Agiliti user manual in effect at the time of sale of the product, including without limitation the safety instructions and if applicable the safe working load and weight limitations set forth therein. If there is no period of time specified below, the warranty period shall be ninety (90) days. Unless agreed to otherwise in writing by Sizewise Manufacturing, the warranty periods commence on the invoice date of the original purchase. This warranty applies only against defects discovered within the warranty period and extends only to you, the Buyer. As used in this warranty, Buyer means the original purchaser or original end user of the product designated at the time of purchase. Any reference to "you" is as the Buyer. Parts repaired or replaced under the terms of this warranty will be warranted for the remainder of the original warranty period only. Buyer is solely responsible for determining whether the products and services offered by Sizewise Manufacturing are appropriate for their intended use including use by Buyer's employees, invitees, contractors, representatives, third party beneficiaries, or any other person Buyer intends to use the product. Buyer acknowledges it has made the selection of the products based upon its own judgment and that the product is of a size, design, capacity, condition, quality, and durability selected by Buyer. Buyer expressly disclaims any reliance upon any statements or representations made by Sizewise Manufacturing or any other party on its behalf except as otherwise specifically provided for in this Limited Product Warranty. Buyer acknowledges and agrees that the remedy in this warranty for the repair and replacement of defective product or part(s) shall constitute the sole remedy in the event of a defective product or part(s).

THE WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS THAT VARY FROM STATE TO STATE. Sizewise Manufacturing's obligations under this warranty are limited as set forth below.

Warranty Period and Coverage:

Product	Internal Components	Top Cover	Core Shield™	RF-Welded Seams
NP Adjust	5 Years	4 Years	4 Years	4 Years

Limited Product Warranty (cont.) D-L00-001 Rev. 5

Conditions and Restrictions

The warranty applies to this product only while it remains in the possession of the Buyer and proof of purchase is demonstrated. Further, the warranty only applies so long as the product has not been subjected to accident, misuse, abuse, improper service, or modification.

This warranty is valid only in accordance with the conditions set forth below:

- Any warranty claims must be made within the warranty period.
- This warranty does not apply to any other products or devices sold separately regardless of whether used with this product. Such other products or devices, to the extent sold with a warranty, will be subject to the separate limited product warranty set forth in the user manual for such product or device.
- Sizewise Manufacturing's sole liability shall be discharged by replacing or repairing, at Sizewise Manufacturing's option, the product or its part or parts which are determined by Sizewise Manufacturing to be defective under normal and proper use during the warranty period.
- Buyer shall notify Sizewise Manufacturing or the authorized Sizewise Manufacturing dealer immediately but in no event later than seven (7) days after the date of discovery of any alleged defect by contacting Sizewise Manufacturing Parts and Service at 800-814-9389, Monday through Friday 8am–5pm local time.
- If the product or part should be returned to Sizewise Manufacturing, a return authorization number (RA#) must be obtained by Buyer from Sizewise Manufacturing. The RA# will be valid for 21 days from the date it is issued.
- Buyer is responsible for any shipping, freight, handling, pickup, or delivery charges or fees, including without limitation, any expediting fees involved with the delivery of the defective product or parts to Sizewise Manufacturing's factory for repair or replacement.
- If on-site technical service is required, as determined by Sizewise Manufacturing, a service representative will be dispatched during Sizewise Manufacturing's standard service hours Monday through Friday 8am–5pm local time, provided the product is located within Sizewise Manufacturing's service territory.
- If Sizewise Manufacturing determines the problem with the product or part(s) is a result of defective material or workmanship, the product or part will be replaced or repaired at the discretion of Sizewise Manufacturing, and at no charge to the Buyer; however, this is subject to the limitations and exclusions of this Limited Product Warranty.
- At the election of Sizewise Manufacturing, replacement parts may be new or refurbished; Sizewise Manufacturing reserves the right to substitute materials if original materials are no longer available.
- If Sizewise Manufacturing determines the product or part that Buyer has requested warranty services on are not covered by the warranty for any reason including, without limitation, because it is outside of the warranty period, excluded from the warranty, or the warranty is void, Buyer shall pay for the repair or replacement services, including parts and labor performed by Sizewise Manufacturing at Sizewise Manufacturing's prevailing time and material rates plus freight and delivery.
- If Buyer declines the repair or replacement service upon notice from Sizewise Manufacturing that it is not covered under warranty, Buyer shall reimburse Sizewise Manufacturing for all costs from investigating and responding to Buyer's request.
- Any costs to Buyer as referred to herein shall be at Sizewise Manufacturing's prevailing time and material rates plus any freight and delivery charges incurred by Sizewise Manufacturing.
- Any assistance provided by Sizewise Manufacturing outside the terms of this warranty does not waive the limits of this warranty.
- Sizewise Manufacturing does not pay labor outside the United States.
- Any description of Sizewise Manufacturing's products is for identification purposes only and is not an express warranty.
- Loaned products, demo or sample products, and/or consumable products including without limitation batteries and sheets are furnished to Buyer on an "AS IS WITH ALL FAULTS" basis.

Exclusions and Limitations

This Limited Product Warranty shall not apply to the below listed events, occurrences, actions and/or items. Sizewise Manufacturing shall have no obligation to make repairs, replace, or correct products including any part or parts of the product as the result of Sizewise Manufacturing's determination of any of the following:

- Software (PROM (Programmable Read Only Memory)) or firmware version upgrades or updates or any other changes and/or any outages, interruptions, delays, attacks, malware, or any other technology or

Limited Product Warranty (cont.) D-L00-001 Rev. 5

- viruses on such software or firmware or Buyer's network, if applicable to this product.
- Normal wear and tear of the product including, without limitation, normal discoloring, body impressions on support surfaces or loss in some resiliency, if applicable to this product, and/or any cosmetic items, consumable items including, without limitation, mattresses, casters, sheets, handsets, and batteries as these items are not covered by this warranty.
 - Damage due to improper transport, storage, installation, maintenance, use, repair, or failure to follow Sizewise Manufacturing's instructions or procedures as detailed in the user manual.
 - Damage or malfunctions resulting from work performed by service providers not authorized by Sizewise Manufacturing.
 - Repairs performed on an Agiliti product or parts missing a serial number or with a serial tag that has been altered, tampered with, or defaced in any manner.
 - Service calls to correct installation of the product, unless installed under contract by Sizewise Manufacturing or its partners and with regard to installation, the terms of the service contract shall only apply to service installation corrections, not this warranty.
 - Shipping, freight, handling, pickup, and delivery charges or fees involved with delivery of the products and or part(s) to Sizewise Manufacturing's factory for repair or replacement and returning the replacement or repaired products and/or parts to Buyer.
 - Any labor costs incurred beyond the applicable labor warranty period.
 - Damage or product failure from causes external to the product or part(s) including, without limitation, power or electric failure or surges, electrical wiring not in compliance with electrical codes, or Sizewise Manufacturing user manual specifications.
 - Damage caused by failure to provide reasonable and necessary maintenance as outlined in the user manual.
 - Damage caused by the use, misuse, negligence, loss, or abuse of the product or any parts by Buyer, including without limitation any third-party beneficiaries, end user, caregivers, patients, or any others that Buyer intends to use the product, including, without limitation, (except Sizewise Manufacturing or an authorized Sizewise Manufacturing service provider):
 - exceeding any specified weight limitations in any product documentation such as the user manual and, including without limitation as applicable to the product, the Safe Working Load, Maximum Patient Weight, and/or Maximum Load as those terms are defined in the product documentation, user manual and by applicable regulations,
 - to the extent the product specifies a minimum load/weight criteria including without limitation a Minimum Patient Weight in order for it to function properly, then any use not in compliance therewith.
 - cleaning upholstery or fabrics with harsh chemicals, or bleach, outside the recommended cleaning guidelines,
 - altering, tampering with, or modifying in any manner without the express written consent of Sizewise Manufacturing any part(s) or structural components or appurtenances of the products,
 - use of the product or part(s) in any manner for which it is not designed or in any manner inconsistent with the information set forth in the Sizewise Manufacturing user manual, including, without limitation, use with other devices, accessories, cables or ancillary products including without limitation inappropriate replacement parts and/or repairs, for which it was not intended.
 - Exposure of the product or part(s) to accident, acts of God, or natural causes (such as natural disasters, fire, flood, wind, water or power failures, or any acts or threats of terrorism, both domestic and foreign) whether foreseen or unforeseen.
 - Operation of the product beyond its normal useful life.
 - Buyer's failure to show proof of purchase.
 - Products or items not manufactured by Sizewise Manufacturing. Rather, for products or items obtained by Sizewise Manufacturing from an original manufacturer or third-party supplier, Sizewise Manufacturing may assign to the Buyer any warranty rights in such products or items that Sizewise Manufacturing may have from the original manufacturer or third-party supplier to the extent such assignment is allowed by the original manufacturer or third-party supplier.

Limited Product Warranty (cont.) D-L00-001 Rev. 5

Disclaimer and Release

The warranties provided herein are the exclusive warranties given by Sizewise Manufacturing and supersede any prior, contrary, or additional representations or warranties, whether oral or written. TO THE EXTENT NOT PROHIBITED BY LAW, THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER WARRANTIES, ORAL, WRITTEN, STATUTORY, EXPRESS OR IMPLIED. EXCEPT FOR THE EXPRESS WARRANTIES IN THIS LIMITED PRODUCT WARRANTY AND TO THE EXTENT NOT PROHIBITED BY LAW, SIZEWISE MANUFACTURING DISCLAIMS ANY AND ALL OTHER WARRANTIES WHETHER EXPRESS OR IMPLIED WARRANTIES, STATUTORY OR OTHERWISE, INCLUDING, WITHOUT LIMITATION, ANY WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR NON-INFRINGEMENT, AND ANY WARRANTIES ARISING FROM COURSE OF DEALING, USAGE OF TRADE, OPERATION OF LAW, OR OTHERWISE WITH RESPECT TO ANY PRODUCT, SERVICES, PARTS INCLUDING REPAIRED OR REPLACED PRODUCTS AND PARTS ARE HEREBY DISCLAIMED AND EXCLUDED. SIZEWISE MANUFACTURING ALSO HEREBY DISCLAIMS AND EXCLUDES ALL OTHER OBLIGATIONS OR LIABILITIES, EXPRESS OR IMPLIED, ARISING BY LAW OR OTHERWISE, WITH RESPECT TO ANY NONCONFORMANCE OR DEFECT IN ANY PRODUCT OR PART(S), INCLUDING BUT NOT LIMITED TO: (A) ANY OBLIGATION, LIABILITY, RIGHT, CLAIM, OR REMEDY IN TORT, WHETHER OR NOT ARISING FROM THE NEGLIGENCE OF SIZEWISE MANUFACTURING OR ITS SUPPLIERS (WHETHER ACTIVE, PASSIVE, OR IMPUTED); AND (B) ANY OBLIGATION, LIABILITY, RIGHT, CLAIM, OR REMEDY FOR LOSS OF OR DAMAGE TO ANY PRODUCT OR PART(S). This disclaimer and release shall apply even if the express warranty set forth above fails of its essential purpose. SOME STATES DO NOT ALLOW DISCLAIMERS OF IMPLIED WARRANTIES, SO THIS DISCLAIMER MAY NOT APPLY TO YOU. TO THE EXTENT SUCH WARRANTIES CANNOT BE DISCLAIMED UNDER THE LAWS OF YOUR JURISDICTION, SIZEWISE MANUFACTURING LIMITS THE DURATION AND REMEDIES OF SUCH WARRANTIES TO THE DURATION OF THIS EXPRESS LIMITED WARRANTY.

Exclusive Remedies

For any product described above that Sizewise Manufacturing determines to have failed to conform to this limited warranty, Sizewise Manufacturing will provide, at its option, one of the following:

1. Repair.
2. Replacement; or
3. Refund of the purchase price.

Sizewise Manufacturing Limited Product Warranty service may be obtained by contacting Sizewise Manufacturing or the authorized dealer from whom Buyer purchased the item. Sizewise Manufacturing compensates only Sizewise Manufacturing-authorized service providers for warranty trips within their normal service area to repair commercial products at the customer's location.

THESE SHALL BE THE SOLE AND EXCLUSIVE REMEDIES OF THE BUYER FOR ANY BREACH OF WARRANTY.

Exclusion of Consequential and Incidental Damages

SIZEWISE MANUFACTURING AND/OR ITS SUPPLIERS SHALL HAVE NO OBLIGATION OR LIABILITY, WHETHER ARISING IN CONTRACT (INCLUDING WARRANTY), TORT, OR ANY OTHER LEGAL THEORY (INCLUDING ACTIVE, PASSIVE, OR IMPUTED NEGLIGENCE AND STRICT LIABILITY), OR OTHERWISE, FOR DAMAGE TO THE PRODUCT INCLUDING PART(S), PROPERTY DAMAGE, DEATH, PERSONAL INJURY, LOSS OF USE, GOODWILL, REVENUE OR PROFIT, COST OF CAPITAL, COST OF SUBSTITUTE PRODUCT, ADDITIONAL COSTS INCURRED BY BUYER (BY WAY OF CORRECTION OR OTHERWISE), OR ANY OTHER INCIDENTAL, SPECIAL, INDIRECT, COMPENSATORY, OR CONSEQUENTIAL DAMAGES, INCLUDING, WITHOUT LIMITATION, WHETHER RESULTING FROM NONDELIVERY, USE, MISUSE, OR INABILITY TO USE THE PRODUCT,

SERVICES OR PART(S). This exclusion applies even if the above warranty fails of its essential purposes and regardless of whether such damages are sought for breach of warranty, breach of contract, negligence, or strict liability in tort or under any other legal theory. SIZEWISE MANUFACTURING LIABILITY SHALL under no circumstance exceed THE AMOUNT PAID BY BUYER FOR THE RELEVANT defective PRODUCT. SOME

Limited Product Warranty (cont.) D-L00-001 Rev. 5

STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION AND EXCLUSION ON SUCH MAY NOT APPLY to you.

Extended Warranty

If the product covered under the Limited Product Warranty set forth herein had from Sizewise Manufacturing an extended warranty option made available only at the time of the Buyer's original purchase from Sizewise Manufacturing, Buyers who then also purchased such extended warranty, and have proof of such purchase, may extend the warranty period stated above in the Limited Product Warranty on parts, electronics, frame, and labor relating to parts, electronics, and frame repairs, as applicable, for an additional one (1) or two (2) years, whichever extension is purchased by Buyer ("extended warranty period"). Other than the extension on the warranty period, all other terms of the limited product warranty including without limitation the conditions and restrictions, exclusions and limitations, disclaimer and release, and exclusion of consequential and incidental damages of the original limited product warranty apply during the extended warranty period. For the purpose of clarity, an extended warranty is not available where the original warranty period is less than one (1) year, and an extended warranty may only be purchased by Buyer at the time of the original purchase of the product from Sizewise Manufacturing.

To make a warranty claim, contact:

RAYE'S, INC. DBA SIZEWISE MANUFACTURING
PO BOX 320
ELLIS, KS 67637

800-814-9389 Monday through Friday 8am-5pm local time

Complete this portion and keep for your records.

Purchased from: _____

Product/model: _____

Serial number: _____

All specifications, equipment, and prices are subject to change without notice. Photos and drawings are representative of the products and may vary slightly from actual production models. All content Copyright Agiliti Health, Inc.

Raye's, Inc., d/b/a Sizewise Manufacturing (f/k/a Wheelchairs of Kansas and Sunflower Medical), is a wholly owned subsidiary of Sizewise Rentals, LLC. Sizewise Rentals, LLC is a wholly owned subsidiary of Agiliti Health, Inc.

For service or support, call 800-814-9389.

